

## Thermoforming the Quality Liner



1. Spray the outer surface of the Quality Liner with alcohol or apply talcum powder.



2. Turn the Quality Liner inside out.



3. Place the bottom of the Quality Liner against the distal end of the positive plaster.



4. Slide the Quality Liner over the positive plaster.



5. Check that the distal end of the Quality Liner is correctly positioned. This is especially important if using a Quality Locking Liner.



6. Place the positive plaster with the Quality Liner in an oven set at 80C, (175F) for a period of 15 minutes for the 1 layer liner or 30 minutes for the 3 layer liner.

# CENTRI®



**7.**  
Remove the positive plaster with the liner from the oven and let it cool.



**8.**  
Spray the outer surface with alcohol or talcum powder.



**9.**  
Remove the Quality Liner by turning it inside out and pulling.



**10.**  
Trim the proximal end of the Quality Liner to desired shape.  
  
Avoid chipping the liner.



**11.**  
The Quality Liner will conform better to the cast the longer over time utilised.  
Recommended:  
15 min for 1 layer liner  
30 min for 3 layer liner