


ALPHA INFANT HAND

Right and Left Models, 3 Colors

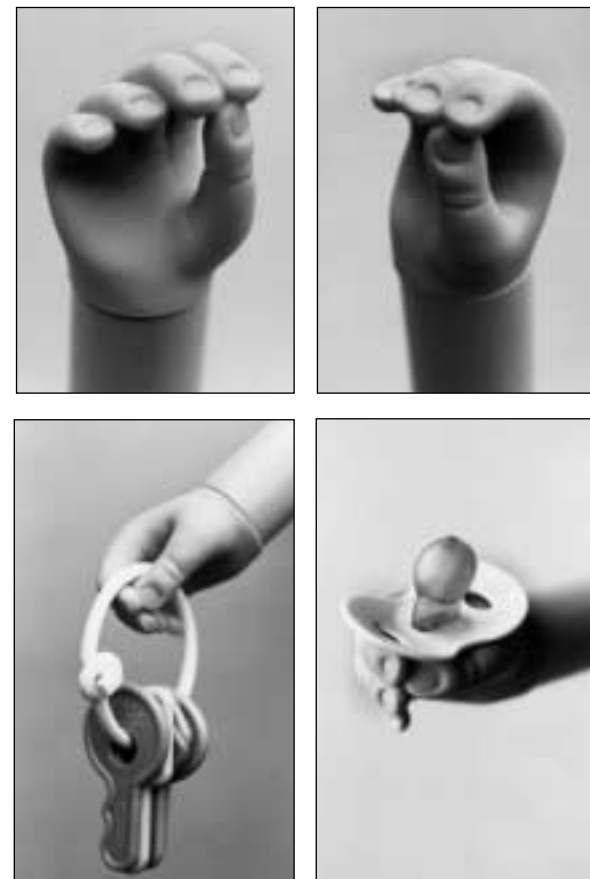

Patent # 5,085,665



PROSTHETICS RESEARCH ■ DESIGN ■ MANUFACTURING ■ CONSULTING

3090 STERLING CIRCLE, STUDIO A, BOULDER, CO, USA 80301-2338

PHONE: 303.444.4720 TOLL FREE: 800.279.1865

FAX: 303.444.5372

ON THE WEB: www.oandp.com/trs

APRIL 2002

 TRS recommends sterilizing these products before use.

Pacifier uses LATEX nipples. Do not use pacifiers if infant or family members are latex sensitive.

Limited Warranty

Products manufactured by TRS, Inc. are covered by this written Limited Warranty for a period of one year from the date of purchase. We encourage all customers (end users) to fill out completely and return the Warranty Card to TRS, within thirty days from the date of purchase. *Product warranty information provides us with another means of tracking which exact product you are using for repair and parts replacement purposes.*

This Limited Warranty covers any defects in workmanship or materials by TRS, Inc. for the period stated, provided the product is used in a reasonable manner. The Limited Warranty is not applicable to normal wear and not applicable to spring or spring-like components. The Limited Warranty applies only to the end user/purchaser who completes the registration card and is not transferable.

Any implied warranties, including merchantability, are also limited to one year from the date of the original TRS Invoice and any action must be commenced within that period. This Limited Warranty is void if the product is misused or abused, if unauthorized alterations are made, or if maintenance is neglected.

TRS, Inc. shall not be liable for any incidental or consequential damages, unless otherwise required by law. Some states do not allow the exclusion or limitation of incidental or consequential damages or the limitation on the period the implied warranty lasts so that the above limitation or exclusion may or may not apply to you. This warranty gives you specific legal rights, and you may also have other rights, which vary from state to state.

Product Serial #: _____

Installation:

The ALPHA Infant Hand is supplied with two options for mounting. The hand has a 1/2 inch diameter threaded receptacle molded into its base. The receptacle can be used alone if a standard wrist unit is not desired. Terminate the prosthesis in a 1/2 inch diameter threaded stud and the ALPHA Infant Hand can be screwed down upon it. Additionally TRS provides a 1/2 inch diameter threaded stud. The stud fits into the receptacle to provide a standard USA style wrist mounting.

Training/Education Information:

The ALPHA is a simple but highly functional prosthetic device. It is flexible, durable and easy to use. The hand is designed for crawling and balance, bi-manual holding and uni-lateral holding options. It provides a safe, flexible interface at the end of the prosthesis. The ALPHA's thumb is attached to the finger tips to create an elastic opening. Safe, age appropriate objects can be inserted into this opening to help the child learn that a prosthesis can be used for holding. TRS provides a pacifier which will "force fit" into the opening. The pacifier and opening are sized to provide a secure fit so that the pacifier will not easily fall from the hand's grasp. If the fit is excessively tight the ALPHA Hand, can be removed from the child and placed into hot water. The hot water will soften the hand slightly making it easier to insert the pacifier.



Never place objects into the hand that could cause injury.

Never insert hard or sharp objects into the hand. The child may fall upon the hand and prosthesis or bring the hand to its face or near its eyes.

Always use caution and always supervise object handling.

No formal training regime exists. Consult with a Registered Occupational Therapist (OTR) or Registered Physical Therapist (RPT) with specific experience in upper extremity prosthetics for specialized help and training.

Care and Cleaning:



TRS recommends sterilizing the ALPHA Infant Hand and pacifier before initial use. Sterilize by placing in boiling water for one to two minutes.

Note: The pacifier nipple is constructed of latex! If your child or family members are latex sensitive do not use the pacifier.

The ALPHA Infant Hand is constructed of a durable polyurethane material. Polyurethane is tough, abrasion and chemical resistant. It resists dirt and stains but permanent dyes will discolor the material.

The ALPHA Hand is designed expressly for infant use. The child will most likely place the hand in its mouth. Always keep the hand and pacifier clean. Clean with mild soaps or detergents and always rinse the ALPHA Hand and pacifier thoroughly to remove all cleaning product residues. Re-sterilize the ALPHA Hand and pacifier as necessary.